

Interlocking Steps

INSTRUCTIONS FOR USE

Code 4188




CONTENTS	PAGE
1.0 INTRODUCTION	1
2.0 PRODUCT CODES AND TECHNICAL DATA	2
3.0 FOR YOUR SAFETY	2
4.0 ASSEMBLY AND USE	3
5.0 CARE & MAINTENANCE	3
6.0 GUARANTEE & SERVICE	4
7.0 CONTINUOUS IMPROVEMENT	5
8.0 WARRANTY & AFTERSALE	6

1.0 INTRODUCTION

Thank you for choosing your new Smirthwaite Product.


Our Interlocking platforms/steps form an adjustable set of steps, each at different heights. Steps can be locked in position to set specific distances between steps, with the lower step being 125mm, the middle step being 250mm and the top surface being 375mm from floor level.

	<p>IMPORTANT!</p> <p>These instructions should be read by all therapists and carers using the equipment and should be retained for future reference. The product should always be used under adult supervision.</p> <p>Any incorrect use of the product and failure to follow the instructions may put the user at risk or impede the function. If you have any queries using this product or wish for further copies, please do not hesitate to contact Customer Service department on T: +44 (0) 1626 835552</p>
---	---

2.0 PRODUCT CODES AND TECHNICAL DATA

Step Size		1	2	3
Product Code		4188		
Width	mm	625	625	625
Depth	mm	370	370	370
Height	mm	125	250	375

3.0 FOR YOUR SAFETY

	STOP!
	Please read these instructions CAREFULLY and THOROUGHLY


- The user should NOT be left unattended whilst using the product. Always ensure a responsible adult or carer is in attendance.
- In the early stages of use, assist the child to walk correctly from behind.
- Regular maintenance checks and cleaning are essential for the safe use of this equipment (see care and maintenance section).
- If any part is loose, damaged or functioning incorrectly, do not use until rectified.
- This product is intended for indoor use only.
- Always keep this product away from naked flames, cigarettes and sources of heat including open fireplaces, radiators, heaters.

If you believe this product to be faulty - **DO NOT USE** - Contact G&S Smirthwaite Ltd on T: +44 (0) 1626 83552.

4.0 ASSEMBLY AND USE

- The product is supplied fully assembled.
- Should you wish to query any aspect of your product, please contact G&S Smirthwaite Customer Services on T: +44 (0) 1626 835552 or info@smirthwaite.co.uk
- The three steps are designed such that they can be interlocked into the next step size up, i.e. the smallest interlocks with the middle size and the middle size with the largest.
- Each step depth can be adjusted, and is achieved by simply removing the handwheels located on either side of the steps. When the desired step depth has been selected, simply re-insert the handwheels, ensuring they are fully tightened before use.

5.0 CARE AND MAINTENANCE


	IMPORTANT!
	Cleaning is recommended on a regular basis

All our wooden products are easy to clean.

- Clean wooden parts with a damp cloth and mild detergent
- Stubborn marks on wood work should be cleaned by using a soft brush.
- Do not soak or immerse the product in water.
- Store the product in a cool dry place out of direct sunlight.
- DO NOT use bleach, solvents, abrasives, synthetic detergents, wax polishes, antibacterial sprays or wipes.
- For further information please refer to MHRA or your NHS cleaning guidelines.

5.1 DAILY CHECKS

- Check all parts for signs of wear and tear or damage.
- Ensure all screws are present and tightened

	STOP!
	The product should not be in use while the checks are carried out.

5.2 SERVICE INTERVAL

Interlocking steps should be serviced every 3 years. Servicing must only be undertaken by a Smirthwaite service engineer, or by a Smirthwaite trained representative.

6.0 GUARANTEE & SERVICE

This product is issued with a full parts and labour guarantee for 2 years from the date of delivery. This guarantee does not apply to accidental damage caused through inappropriate use of the product. This guarantee is issued at the discretion of G&S Smirthwaite Ltd on RTB (Return to Base) basis. If you have any questions or doubts relating to the safety or use of the chair please contact us for advice or assistance on:

T: +44 (0) 1626 835552

E: info@smirthwaite.co.uk

7.0 CONTINUOUS IMPROVEMENT

G&S Smirthwaite Ltd are committed to continuous improvement to their product range. Should you have any suggestions or comments please send them to our product design department at: info@smirthwaite.co.uk

G&S Smirthwaite Ltd reserve the right to change the specification or material without prior notice.

For catalogues, help and further information on our products please contact us at:
G&S Smirthwaite Ltd, 17 Wentworth Road Heathfield, Devon TQ12 6TL

T: +44 (0) 1626 835552

F: +44 (0) 1626 835428

E: info@smirthwaite.co.uk

W: www.smirthwaite.co.uk



G&S Smirthwaite Ltd, 17 Wentworth Road, Heathfield, Newton Abbot, Devon.TQ12 6TL
T: +44 (0) 1626 835552 F: +44 (0) 1626 835428 E: info@smirthwaite.co.uk
www.smirthwaite.co.uk

