


## T ROLL OPTIONS

The snooooooze T Roll's soft profile allows moulding to the clients shape and therefore gives greater support. It's important the feet present well to the bed surface and the legs are well aligned with a good pelvic position. An indicator of a poor postural position is illustrated with the legs 'hugging' the pommel, as 'right'. Here we have potential rubbing of the legs/feet and the hips are outwardly rotated.

We can address **ADDUCTION** at the knees, as the snooooooze T Roll has two independent inner, we substitute the pommel's soft foam inner for a solid piece wrapped in pressure distribution foam.

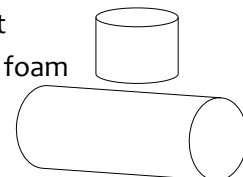

**PLEASE NOTE THE CURVED T-ROLL IS A SINGLE INNER. WE CAN AS A SPECIAL ORDER PRODUCE IT WITH A SOLID POMMEL.**



The curved T Roll configuration came from a demand to allow the client to easily reposition from supine to side, the reduced size of the ends allows the bottom leg to be aligned correctly and the wrap holds the legs in position. Often supplementary support is needed for the upper foot.

The Double Pommel T Roll is a wonderful configuration that adds the ability to give positioning to the lower legs and feet.




For some clients the standard T Roll will allow some supine to side repositioning. Often supplementary support is needed, as is some support for the upper foot.

**ASK FOR DETAILS ABOUT  
PRESSURE MANAGEMENT  
THERMAL REGULATION  
W LEG TROUGH OPTIONS**



# snoozoze®


POSITION. COMFORT. REST.

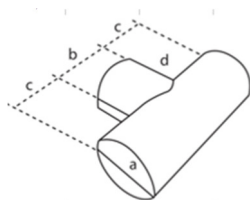

ALL SIZES IN mm


	min	paed	std	max
a	100	150	250	400
b	100	150	250	400
c	100	225	250	400
d	100	150	250	400

PRODUCT CODE, ROLL/T & ROLL/PA (Standard T Roll)  
Visco-Elastic Crumb (Memory Foam) fill with Removable Cover

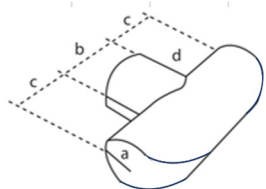

	min	std	max
a	50	125	200
b	100	250	400
c	100	250	400
d	100	250	400

PRODUCT CODE, ROLL/T/C (Curved T Roll)  
Visco-Elastic Crumb (Memory Foam) fill with Removable Cover

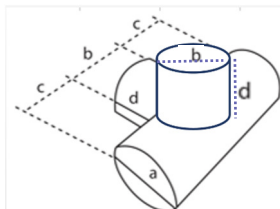


	min	std	max
a	100	250	400
b	100	250	400
c	100	250	400
d	100	250	400

PRODUCT CODE, ROLL/T/DP (Double Pommel T Roll)  
Visco-Elastic Crumb (Memory Foam) fill with Removable Cover



PRODUCT CODE add '/WRAP' to existing code

VISIT OUR YOU TUBE CHANNEL TO SEE ALL  PRODUCTS



<https://www.youtube.com/channel/UCa15hZ5KgYHHWrtlaZiLzcA/feed>



Benfield Business Park | Benfield Business Road | Newcastle upon Tyne  
NE6 4NQ | United Kingdom

POSTURECARE DESK +44 (0) 191 276 9636/7

t +44 (0) 191 276 9600 f+44 (0) 191 276 9696

For more details please email:

[allison.mcgreevy@peacocks.net](mailto:allison.mcgreevy@peacocks.net) / [roger.somerville@peacocks.net](mailto:roger.somerville@peacocks.net)

For patient/client identifiable information please use email:

[allison.mcgreevy@nhs.net](mailto:allison.mcgreevy@nhs.net)