PROTAC®

INTEGRATING SENSES

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GB - Directions for use

Protac Ball Cushion®

You have selected a quality Protac product. Protac guarantees that you have purchased an original, Danish designed product developed in consultation with experts. Protac has developed, produced and sold sensory-stimulating aids to the healthcare sector in Denmark, the rest of Europe, Australia and New Zealand since 1994. Protac products are developed on the basis of theories on sensory integration, and are supported by extensive research, documentation and case studies. Protac focuses strongly on protection of the environment and safety during its production process. Protac plastic balls are certified according to STANDARD 100 by OEKO-TEX® no. 2076-310 DTI.

For more details, go to www.protac.dk

GB

Labelling

The products are labeled in accordance with Regulation (EU) 2017/745 Class I medical devices. The label with details of manufacturer, raw materials and washing instructions is sewn into the side of the cushion.

Key to label symbols: please refer to Key to symbols

General precautions - Safety

- ⚠ • The cushion must be securely fastened to a chair
- ⚠ • Please be aware of how the user reacts
- ⚠ • Not suitable for wheelchair-users
- ⚠ • Not for unsupervised use by users with paralysis and poor sensitivity.
- ⚠ • Must not be used near an open flame

Product purpose and areas of use

Sensory integration is the idea behind the development of all Protac products. The theory originated in the 1970s in the USA, and was developed by A. Jean Ayres, an occupational therapist and psychologist. The theory is based on how the brain processes sensory impressions, and how the various sensory stimuli can be influenced via applying weight and pressure to the tactile sense (the sense of touch) in the skin and the proprioceptive sense (the sense of movement) in muscles and joints. The pressure, movement and weight of the balls act on these senses in particular.

The Protac Ball Cushion® was developed to help sufferers of function impairment.

Children, young and old people can benefit from the cushion:

- Brain damage, including spasticity, Huntington's Chorea, Parkinson's Disease and other ailments
- ADHD and autism
- Psychiatric diseases, such as dementia, schizophrenia, depression, anxiety, anorexia etc.
- Development problems, sensory problems, deaf/blind, etc.
- Stress

The Protac Ball Cushion® is intended for patients with brain damage suffering from spasticity and sensitivity impairment. The weight and pressure of the balls help suppress spasticity to give the patient enhanced body awareness. It can also be used to train balance with.

Individuals with ADHD and autism can use the Protac Ball Cushion® to sit on e.g. in school and at home when unconcentrated. When the user moves on the cushion, the pressure of the balls stimulates muscles and joints, helping improve body posture.

Psychiatric patients often suffer from muscular and psychological unrest. The cushion can be used to enhance concentration and help calm the patient. The cushion can be used for meetings and dining.

The Protac Ball Cushion® is used for sufferers of a range of development problems, sensory problems, deaf/blindness and to create enhanced sensory stimuli in the body.

The Protac Ball Cushion® is used for sufferers of stress to provide calmness, comfort and better body awareness.

GB

The Protac Ball Cushion® can be used in institutions, sheltered housing, day centres, hospitals, patient lounges and private homes. The cushion is used in schools in the classroom for instance, and at home, can be used on dining or office chairs.

Using the product

The cushion must be placed on a hard surface, e.g. a classroom stool or one-piece moulded chair to gain full benefit of sensory stimulation.

The cushion must be securely fastened to a chair.

We recommend ensuring that the child has something to rest his/her feet on, e.g. the floor, a foot stool or similar.

During normal use of Protac Ball Cushion® the child will move on the cushion, therefore, sitting on the balls will not feel hard.

Product service life

The product has an expected service life of 5 years.

This is based on correct use, cleaning, washing, service and maintenance.

Products used beyond the aforementioned service life should be assessed for further use.

Responsibility for using the product beyond the stated service life rests with the user.

Cleaning

Protac Ball Cushion® can be washed. The regularity and need for washing depends on use. See below for washing and cleaning instructions.

Protac Ball Cushion®

- can be machine washed in a min. 6 kg capacity washing machine. Use washing detergent with no bleaching agents or perfume - see also the washing instructions on the cushion.

To remove spots using a spot remover, always try first on an area which is not visible to test for damage or discolouring. Protac does not recommend the use of spot remover.

Protac cannot be held liable for incorrect washing and maintenance.

Maintenance

We recommend airing the cushion regularly.

Transport and storage

The cushion is ready for use.

It is supplied packed in a protective plastic bag.

If you are not using the cushion, store it carefully in its case to protect from damp and dirt.

Disposal

The cushion can be broken down into the following components for disposal: Plastic and textile.

The plastic balls can be disposed of as hard plastic.

All components can be disposed of as combustible refuse.

Always dispose of refuse according to local rules.

Guarantee

2 year guarantee against manufacturing defects.

The guarantee will be invalid in the event of the following:

- over-use
- misuse
- non-maintenance or incorrect maintenance

Traceability: The month, year of manufacturer and batch number are marked on the CE - and care label. These must not be removed from the product.

Key to symbols

Example only. Always follow the recommended care instructions on the label found inside the product

Washing bowl: Maximum washing temperature (see care label)

Warning:
See under Safety

Triangle:
Do not bleach

User information

Square with a circle:
tumble drying possible
(see care label)

For indoor use

Iron:
Do not iron

CE marked according to
(EU) 2017/745 Class I
medical devices

Circle:
Do not dry clean

Manufacturer

Batch number

Technical specifications

Dimensions:

Protac Ball Cushion® for classroom chair: 40x40 cm

Protac Ball Cushion® for baby's high chair: 29x45 cm

Protac Ball Cushion® for kindergarten chair: 30x35 cm

Protac Ball Cushion® for stool: 30x30 cm

Standards: Comply with general requirements for aids acc. to the following harmonized standards:

Clinical evaluation: EN 12182:2012

Risk analysis: DS/EN ISO 14971 : 2012

Quality management system: EN ISO 13485:2016

Ignitability test: EN 1021-1 & 2: 2014

GB

Service information

No other balls can be used in a Protac Ball Cushion® than those used by the manufacturer.

Guide

- to daily inspection of the Protac Ball Cushion® before use

No.	Check	Remarks
1	• All parts are in place, no loose parts	
2	• No sign of holes, damage to the textile or seams	

Protac A/S

Niels Bohrs Vej 31 D, Stilling

DK-8660 Skanderborg

Denmark

Tel.: +45 86 19 41 03

e-mail: protac@protac.dk

Distributor list: Please refer to: www.protac.dk/kontakt

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